

Hold time on your side

HYALURONICA®





10 - year proven range of fillers and biorevitalization!


HYALURONICA® is a range by Vital Esthetique - a French company based in Paris ISO 13485-certified for design, manufacturing and distribution of medical devices.

Vital Esthetique's ranges are high quality products with strong age-defying and skin imperfections improving benefits.

In 2018 **HYALURONICA®** celebrated 10 years of its existence and active representation in the aesthetic field, currently in around 50 countries worldwide!

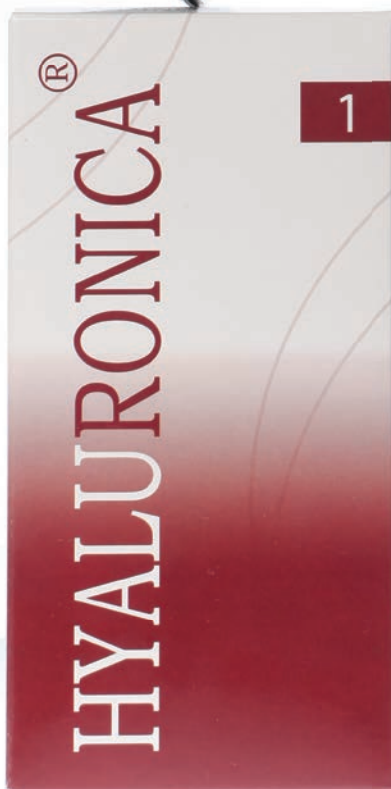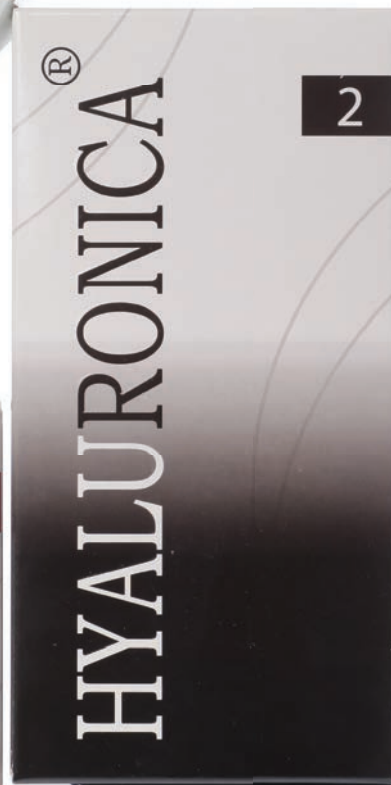

	HYALURONICA® ①	HYALURONICA® ②	HYALURONICA® ③	HYALURONICA® Mesolift
CONTENTS	Cross-linked Hyaluronic acid		High cross-linked Hyaluronic acid	Non cross-linked Hyaluronic acid
PRESENTATION	2 x1.0 ml	2 x1.0 ml	2 x1.0 ml	1 x 1.0 ml
NEEDLES	27G ½	27G ½	27G ½	32G ½
CONCENTRATION	16 mg/ml	23 mg/ml	25 mg/ml	13 mg/ml
INJECTION DEPTH	Upper to Mid dermis	Mid to Deep dermis	Very deep/ subcutis	Superficial dermis
STORAGE	2°C ÷ 25°C			
INDICATIONS	<ul style="list-style-type: none"> - Crow's feet - Perioral wrinkles - Lips contouring 	<ul style="list-style-type: none"> - Cheekbones and cheeks volume - Nasolabial folds - Lips augmentation 	<ul style="list-style-type: none"> - Visco-augmentation - Facial contour 	<ul style="list-style-type: none"> - Biorevitalization: - Face - Neck - Neckline - Hands
LASTS	5 to 6 months	6 to 9 months	9 to 12 months +	–

 **HYALURONICA® range** consists of scientifically studied **filling products - Medical devices, Class III**, which comply with Directive 93/42 EEC, and thus CE-marked.

➤ **The augmentation types of the range are cross-linked: HYALURONICA® ①, HYALURONICA® ② and HYALURONICA® ③** consist of ultra purified Hyaluronic Acid (HA) from a natural and non-genetically modified bacterial stem, and free of animal components. The cross-linker used is BDDE.

➤ **HYALURONICA® Mesolift is based on non-cross-linked** ultra purified Hyaluronic Acid from a natural and non-genetically modified bacterial stem, and free of animal components.

➤ **HYALURONICA® ①, ② and ③** are sterile, biodegradable, viscoelastic, clear, colourless, isotonic, apyrogenic and homogenized injectable gels, which consist of **cross-linked Hyaluronic Acid**, produced by Streptococcus equi bacteria.

➤ **HYALURONICA® Mesolift** is a sterile, biodegradable, viscoelastic, clear, colourless, isotonic, apyrogenic and homogenized injectable gel, made of **non-cross-linked Hyaluronic Acid**, produced by Streptococcus equi bacteria.



Forehead lines ①

Crow's feet ①

Cheekbones and cheeks lines ①

Cheekbones and cheeks volume ②

Remodeling facial contours ③

Fine nasolabial lines ①

Mid to deep nasolabial folds ②

Perioral wrinkles ①

Lip contouring ①

Oral commissures ②

Lip augmentation ②

Marionette lines ②

Chin lines ②

Chin folds ②



HYALURONICA® ①

It is a long-lasting, but non-permanent soft filler, that spreads slowly into the skin to achieve smooth and natural look. It is recommended to be injected into **upper to mid dermis** to correct **mid folds**, such as **smile** or **smoke lines alongside the lips**; perfect for **lip contour** enhancement. Easy to inject and shape.



HYALURONICA® ②

It is a long-lasting, but non-permanent soft filler, that spreads slowly into the skin to achieve smooth and natural look. It is designed to instantly restore **skin's volume** and **facial malformation** into **mid to deep dermis**. It is an excellent choice to treat **nasolabial folds, marionette lines, cheekbones**. Gives marvelous results in **lips' augmentation** and **asymmetry correction**. Easy to inject and shape.




HYALURONICA® ③

It is a long-lasting, but non-permanent filler, based on **high cross-linked HA**. It is designed to be used for **visco-augmentation, volume correction, remodeling facial contours and filling in very deep skin depressions**. It is indicated to be injected **very deep (subcutis)**.



This product is an excellent "instrument" for overall face remodeling, but only in the hands of highly qualified and experienced in volumizing procedures doctors.

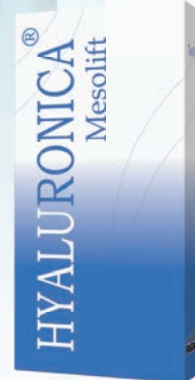

HYALURONICA® Mesolift

Created to be the best supporter of the fillers' procedures, it prolongs the effect of the filling treatments and adds more brightness and moisture to the skin's surface.

It is designed to be injected into the **superficial dermal tissue** and acts as a filler for **small lines like crow's feet**. It is the perfect solution to treat **neck, neckline and hands**.

It is also an excellent choice for **young patients** (20 ÷ 25 years) who don't need augmentation, but need brightness and fresh look, as we all do.



INJECTION TECHNIQUES:

☝ The technique used is essential for the result of the treatment, therefore, it must be injected by a **physician, who has gone through specific injection techniques' training for filling/augmentation procedures.**

➤ **HYALURONICA® ①, ②:** The application can be done by punctual injection (serial punctual technique) or "tunnel injection" (linear threading technique).


➤ **HYALURONICA® ③:** The application can be done by injection of volume depots or by "tunnel injection" (linear threading technique).

➤ **HYALURONICA® Mesolift:** The application can be done by punctual injection (nappage) or by "tunnel injection" (linear threading technique).


☝ **HYALURONICA®** gels have unquestionable safety profile, known and approved during 10 years by hundreds of aesthetic practitioners worldwide. The average durability of the fillers **HYALURONICA® ①, ②** and **③** is 6 to 12 months. The duration period may vary depending on several factors:

- Skin quality
- Lifestyle (smoking and/or exposure to sun, or UVs, may shorten the effect)
- Injection site
- Metabolism



BEFORE & AFTER

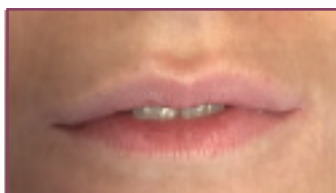

LIPS

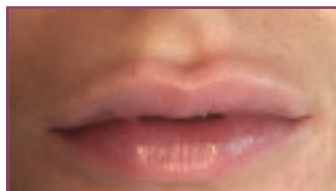

MARIONETTE LINES*

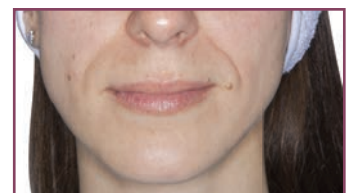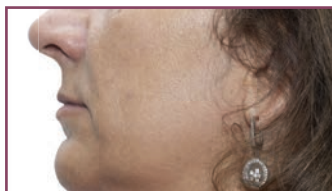

NASOLABIAL FOLDS*



***After the first HYALURONICA® procedure**

It is without a doubt that the fillers, based on Hyaluronic Acid, are the most safe filling treatments, as the Hyaluronic Acid is naturally present in the skin.

An interesting fact is that a human with 70 kg weight has approximately 15 g of Hyaluronic Acid in his body, as 1/3 of that HA endure degenerative and re-synthesis processes every day. Over 50 % of that compound is contained in the skin.

That is why we designed **HYALURONICA®** - to supply your patients with more or to replace the lost HA in their skin. The filling procedure with HA fillers is vital for the hydration of the skin tissue, keeping it fresh and smooth, boosting skin's elasticity, enhancing collagen synthesis, preventing wrinkle formation and promoting skin's firmness.



We thank you for reaching the end of this brochure. Our goal was to make it useful to the point and easy to understand for all professionals around the world.

*We would like to express our appreciation to all of you, who already chose **HYALURONICA®** and are our trusted partners for more than 10 years.*

*To all of you, who just became familiar with our **HYALURONICA®**, we would like to say: "Congratulations, now you have the best "weapon" to fight aging and skin's imperfections!"*

We are proud to deliver to you an optimal range of 4-type products, which are extremely safe, long-lasting and have been approved by professionals during this 10-year period.

*By using **HYALURONICA®** range you can guarantee your patients an immediate visible effect, performed by secure method that will provide natural look and radiant skin appearance.*

If there is a defect on the soul, it cannot be corrected on the face. But if there is a defect on the face, and one corrects it, it can correct a soul.

Jean Cocteau

vital
ESTHETIQUE

CE 1014 ISO 13485

40, Avenue de la Grande Armée
75017 Paris - France
info@vitalesthetique.com
www.hyaluronica.com
www.vitalesthetique.com

Distributed by:

